

ClearView™

Anti-Fog Scope Warmer

Prevents Scope Condensation



Simple Pre-heats the scope to body temperature
Eliminates the cause of condensation

Safe Secures surgeon's endoscopic view
No electricity nor hot water needed

Fast Activated by simply clicking the disk
Scopes are warm for use within minutes
No interruption of surgery for lens defogging

ClearView™ Anti-Fog Scope Warmer,
sterile packed for single use;
10 pcs per Box, order no. **31SW10**
30 pcs per Case, order no. **31SW30**

medispar*
Medical Distribution Partner


CE0197




ClearView™

Anti-Fog Scope Warmer

LAY SCOPE(S) IN WARMER

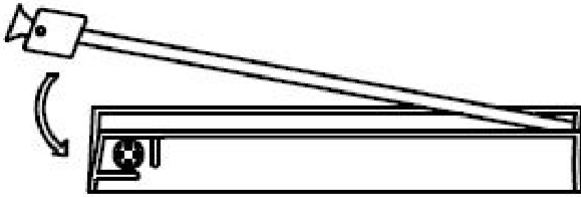

FLEX DISK 2 OR 3 TIMES

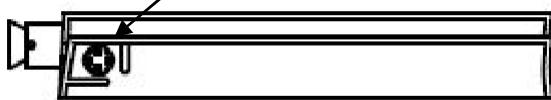

WARM FOR USE IN 2 MINUTES

Note: Best to start the scope warmer 5 minutes before the scope is needed.

*ClearView™ stays warm up to one hour.
Cover with sterile towel to preserve sterility and retain warmth.*

CE 0197
LATEX FREE
DEHP FREE
SINGLE USE
PRODUCT STERILE UNLESS PACKAGE
OPENED OR DAMAGED

Manufacturing date: shown on the seal
Expiration date: 3 years from manufacturing date
Normal storage conditions: not below -2°C



MADE IN PRC FOR 2MEDICAL-EUROPE
Product no. 31SW01



Email: info@medispar.eu
Tel: 011 27 04 17